


EMED Technologies Corporation
1262 Hawks Flight Court, Suite 200
El Dorado Hills, CA 95762, U.S.A.



Emgro Europe
Westervoorsiedijk 60
6927 AT Arnhem
The Netherlands

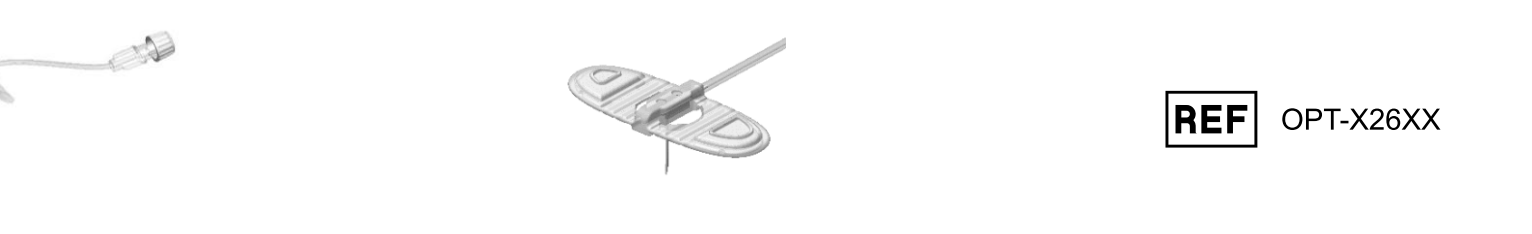

REF OPT-X26XX

Table with 14 columns: ENGLISH (EN), FRENCH (FR), SPANISH (ES), GERMAN (DE), ITALIAN (IT), SWEDISH (SV), DANISH (DA), FINNISH (FI), NORWEGIAN (NO), PORTUGUESE (PT-BR), TURKISH (TR), SERBIAN (SR), POLISH (PL), DUTCH (NL), GREEK (EL), CZECH (CS). Rows include Description, Intended Use, Warning, Features, Specifications, Important Information, and detailed instructions in 14 languages.

Table with 14 columns: ENGLISH (EN), FRENCH (FR), SPANISH (ES), GERMAN (DE), ITALIAN (IT), SWEDISH (SV), DANISH (DA), FINNISH (FI), NORWEGIAN (NO), PORTUGUESE (PT-BR), TURKISH (TR), SERBIAN (SR), POLISH (PL), DUTCH (NL), GREEK (EL), CZECH (CS). Rows 1-15 provide detailed instructions for various steps: 1. Wash hands, 2. Inspect for breaches, 3. Remove the infusion set, 4. Remove the tape from the tubing, 5. Prepare the source of infusion, 6. Remove the Luer lock cap, 7. Connect the female Luer lock, 8. Prime infusion set, 9. Select the site of infusion, 10. Clean site with antiseptic, 11. Prepare the Soft-Site dressing, 12. Remove the wing guard, 13. Fold the wings, 14. Remove needle protector, 15. Use thumb and index finger to pinch the skin.

